

Alivio Bed

Remote Control Tether

KA3200RCTE



No.	Description
1	Tether x 1pcs
2	Adhesive tape x 1pcs


Installation Guide





Step 1: Open packaging, ensure all parts shown above are included.




Step 2: Cut the Adhesive tape to the size of battery lid of your remote control. For the grey remote control, cut adhesive to **34mm**. For the white remote control, cut adhesive to **28mm**.

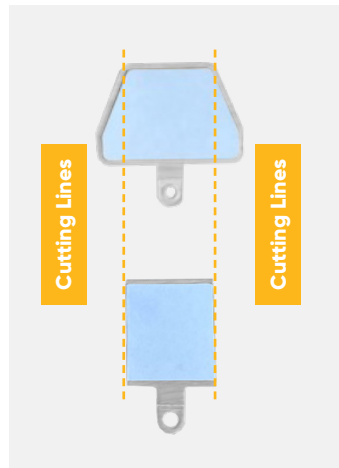

Step 3: Remove the white back of the adhesive, place adhesive at the middle of the battery lid and ensure the tab is at the bottom, firmly press it for 10 seconds.

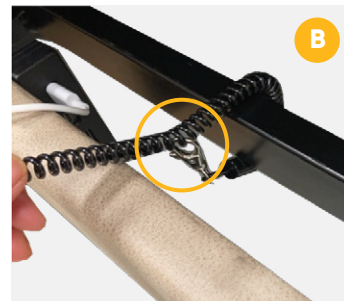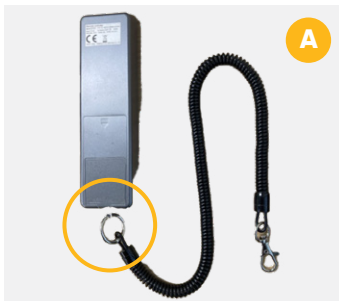

Step 4: Clip the tether ring on the tab of the adhesive (**A**). Place the other end of the tether around any preferred position of the bed frame or bedrail (image shows an example position) and clip the hook on the tether (**B**).

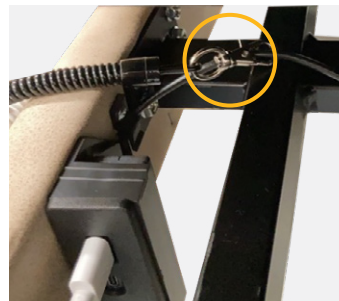

Another option is to clip the hook on the USB cable.

Please contact customerservice@kcare.com.au for more information.

www.kcare.com.au | 1300 783 783