

Alivio Bed Simple Remote Control

KA3200RCS

Battery Installation

1. Remove back lid of the remote control.
2. Install 3 AAA batteries in the battery case.
3. Put the lid back on. Batteries not included.




Blue tooth pairing

The remote control must be paired with the bed before use.


1. Refer to the user manual 3.4 Backup battery 'To disconnect backup battery', so that the power supply unit is connected with the control box directly.

Note: Once the control box is powered, it will automatically enter pairing mode for 30 seconds.

2. Plug power back in and start the next step within 30 seconds.
3. Press and hold **(A)** "SW" button (SWITCH) until the backlit LED light on the remote flashes. Release "SW" and press and hold **(B)** Arrow button Backrest Up until the backlit LED light is lit. You will also hear a two - second beeping from the controller to indicate pairing is completed. Now the remote is paired.
4. Once remote control is paired, reconnect backup battery.

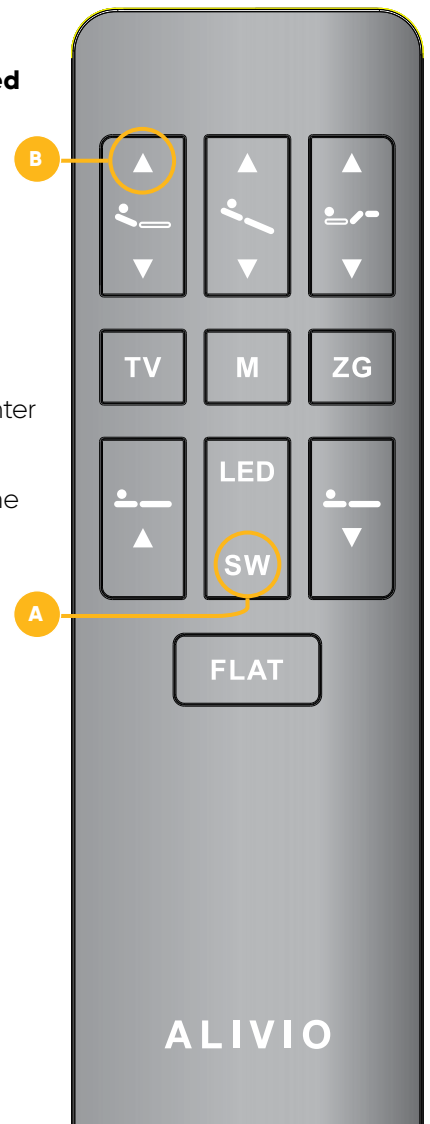


Alternatively, press the pairing button on the control box under the bed until the white light next to the button starts flashing, the control box enters pairing mode. Perform step 3 above to pair the remote with bed.

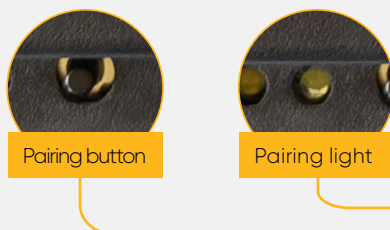

Please contact customerservice@kcare.com.au for more information.

www.kcare.com.au | 1300 783 783

Buttons

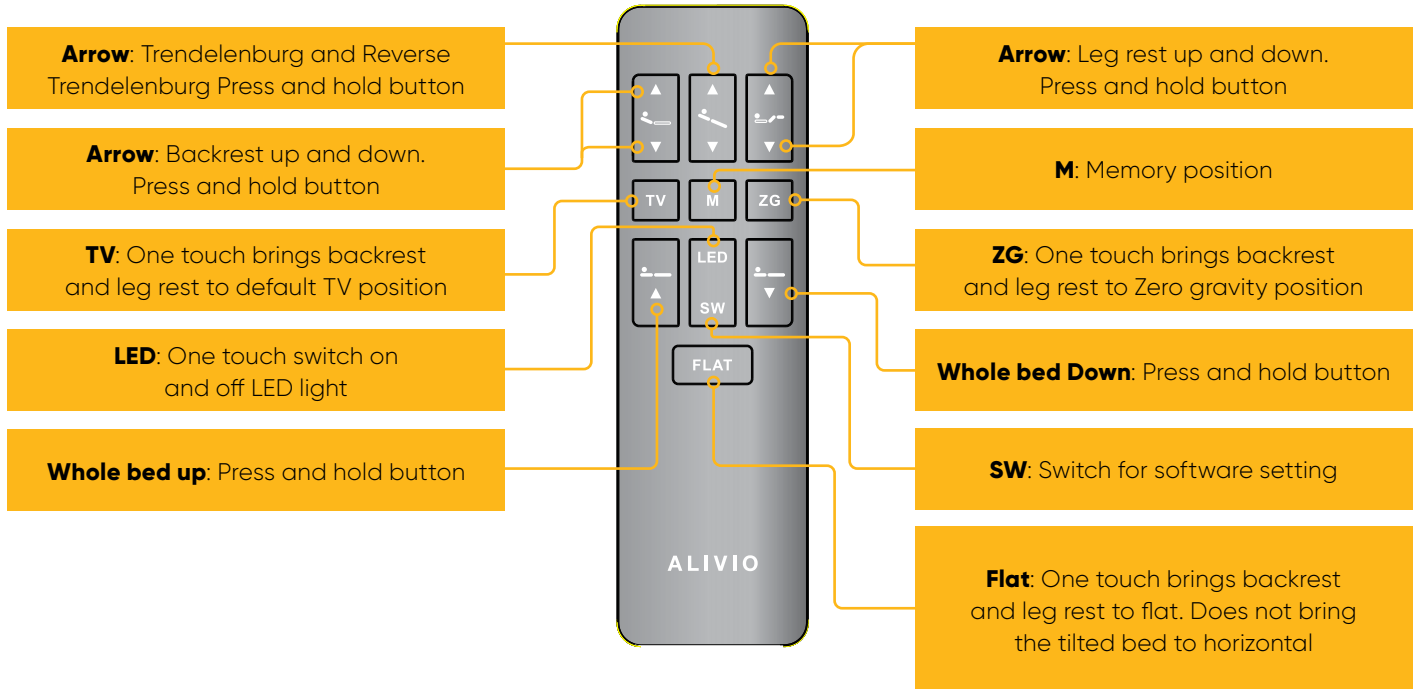

Notes

- The backlight of the remote control lights up for 3-4 seconds when any button is pressed.
- All arrow buttons are Press-and-Hold buttons. The rest are all one-touch buttons.
- When the bed is moving, press any button to stop the bed from moving.
- The wireless control working distance is within 8 meters of the bed. The handset will not work if out of the specified range.
- **Low battery alert:** The backlight of the remote control will flash for 10 seconds when any button is pressed.

Warning: Do NOT keep all castors locked when raising/lowering or tilting the bed. Unlock castors at one end (head or foot end) to allow small movement of castors.

Adjust default positions and set memory position

The '**SW**' (Software setting) button acts as a function button that works with position buttons **TV**, **ZG** and **M** to override the manufacturer pre-setting and memorize your favourite position.

Override position pre-setting

Step 1: Press a nominated pre-set position button and adjust manually to a desired position.

Step 2: Hold **SW** button for 3 seconds until the backlight flashes, and then press and hold the nominated button again until the backlight stops flashing. The original position setting is overridden by the new setting.

Set Memory position

There should be no pre-set memory position out of factory. The bed should not move when **M** button is pressed. Otherwise, cancel the current memory

position first as per "Clear memory position" below, then set the new memory position.

Step 1: Adjust the bed to a position need to be memorized.

Step 2: Press and hold **SW** button for 3 seconds until the backlight flashes, and then press and hold the **M** button until the backlight stops flashing. Memory position is now set.

Clear memory position

To cancel the current memory setting, press and hold **SW** button for 3 seconds until the backlight flashes, and then press and hold the **M** button until the backlight stops flashing. Now the memory position setting is cancelled.

To reset new position: repeat above "Set Memory Position".

Please contact customerservice@kcare.com.au for more information.

www.kcare.com.au | 1300 783 783