


# Jannali Bed

# ALIVIO

## Bed Stick U Shape

KA3200ARMBSU


Installation guide

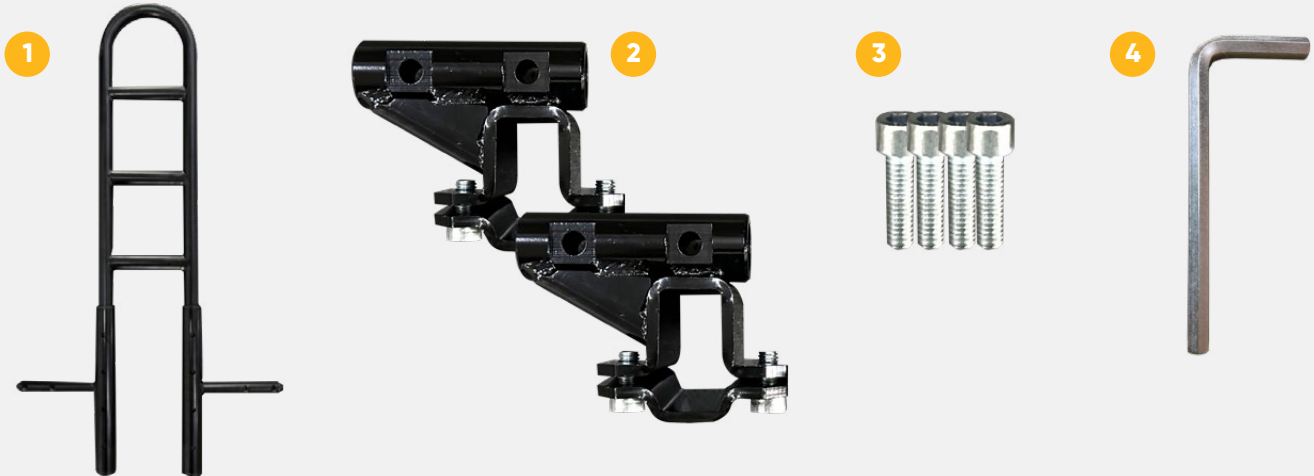

### Installation guide

**Step 1:** Carefully remove all components from packaging. Ensure all parts shown above are included.

Bring the backrest up using remote control to expose mounting rails on the frame.

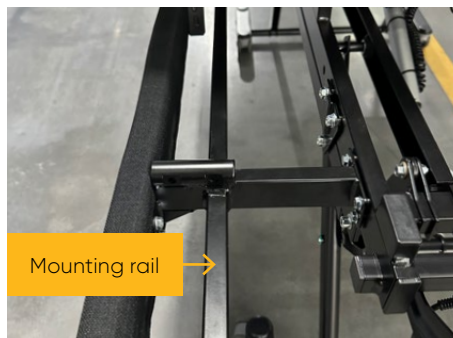

No.	Description
1	U shape bed stick with pre-assembled T-shape mounting bars x 1 set
2	Mobile brackets with screws x 1 pair
3	Screw M8x25mm x 4 pcs
4	Allen key 6mm x 1 pcs

**Step 2:** Preassemble the top half mounting bracket.



**A.** Take the top half of the mounting bracket, slide the tube onto the T-shape mounting bar on the bed stick.



**B.** Place the two screws in the holes without fully tightening them. Do the same to the other side.



**C.** Turn the T-shape mounting bars 90 degrees for installation as image shows.

Sales, administration and general enquiries  
**1300 783 783 | customerservice@kcare.com.au | kcare.com.au**

Published August 2024

**KCare**  
Healthcare Solutions

## Installation guide Continued

# ALIVIO

**Step 3:** Install bed stick on bed.



**A.** Locate the preferred position for the bed stick, place the sub-assembly over the mounting rail.




**B.** Place the bottom half of the bracket under the top half by aligning the holes and place the two screws in the holes without fully tightening them.

**Step 4:** Adjust bed stick height.

**Note:** It is easier to adjust the height before the screws are tightened up.

**Step 5:** Tighten all screws. Lower the backrest of the bed to the flat position.

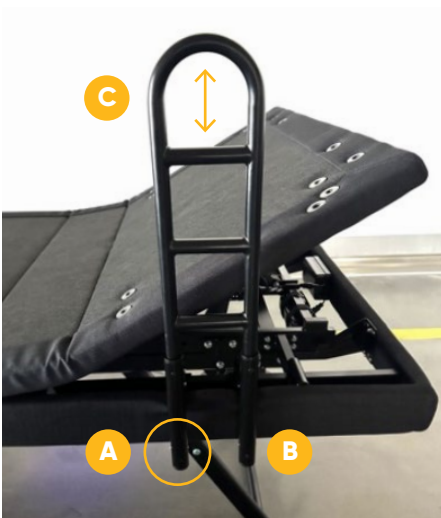

**A.** Press both pins on one mounting tube, move the bed stick a little up or down until the press pins are out of the holes. **B.** Do the same to the other side of the stick. **C.** Hold the top of the stick, move it up or down until pins on both sides are engaged. Ensure both sides are the same height.



## Safe use guide of bed sticks

**Warning:** Bed sticks pose a risk of strangulation and should never be used for people who frequently fall or with cognitive impairment.

Refer to website for full safe user guide

Sales, administration and general enquiries

**1300 783 783 | [customerservice@kcare.com.au](mailto:customerservice@kcare.com.au) | [kcare.com.au](http://kcare.com.au)**

**KCare**  
Healthcare Solutions