

Jannali Bed

Companion Bed

Installation guide



Warning: Ensure safe manual handling practices are in place as the bed is heavy.

Installation guide

Step 1: Open packaging, remove mattress retainer by cutting off the cable ties. ✂

No.	Description
1	Companion bed x 1 pcs
2	Bed leg x 4pcs
3	Mattress retainer x 1pcs



Step 2: Insert bed leg screw in receiving holes on bed frame.



Step 3: Turn and secure bed legs.



Step 4: Turn bed over and lay on the floor, remove packaging



Step 5: Install mattress retainer.

