

Jannali Bed Armrest

Installation guide

ALIVIO



Installation guide

Step 1: Open the packaging carton, prepare the armrest and accessory kit.

Step 2: Use remote control to raise bed to a comfortable position for installation.

Step 3: Install connecting parts.

No.	Description
1	Armrest x 2pcs
2	Armrest connecting part x 4pcs
3	Allen key 5mm x 1pcs
4	Allen key 6mm x 1pcs
5	Screw M10 x 4pcs
6	Metal washers big x 4pcs
7	Screw M8 x 8pcs
8	Metal washers small x 8pcs



Locate the mounting holes on mattress support platform.



Place the armrest connecting part (2) on top of the mounting holes.



Use two sets of M8 screw (7) and washer (8) to fix one connecting part.

Sales, administration and general enquiries
1300 783 783 | customerservice@kcare.com.au | kcare.com.au

Published July 2023

KCare
Healthcare Solutions



Tighten the screws with 5mm Allen key **(3)**.



Mount the other armrest connecting part **(2)** by repeating above steps.

Step 4: Install armrests.



Place the armrest **(1)** on top of the mounting holes on the connecting part **(2)**.



Connect armrest **(1)** with connecting part using two sets of M10 screw **(5)** and big washer **(6)**.



Tighten the two screws with 6mm Allen key **(6)**.

Step 5: Install the other armrest by repeating the above steps.

After installation

