

# Jannali Bed

## Assist Rail Tall

ALIVIO

KA3200ART

Installation guide



## Installation guide

**Step 1:** Open the packaging carton, prepare the assist rail and accessory kit.

**Step 2:** Use remote control to raise bed to a comfortable position for installation.

**Step 3:** Install connecting parts.


No.	Description
1	Assist rail x 2pcs
2	Assist rail connecting part x 4pcs
3	Allen key 5mm x 1pcs
4	Allen key 6mm x 1pcs
5	Screw M10 x 4pcs
6	Metal washers big x 4pcs
7	Screw M8 x 8pcs
8	Metal washers small x 8pcs



Locate the mounting holes on mattress support platform.



Place the assist rail connecting part (2) on top of the mounting holes.

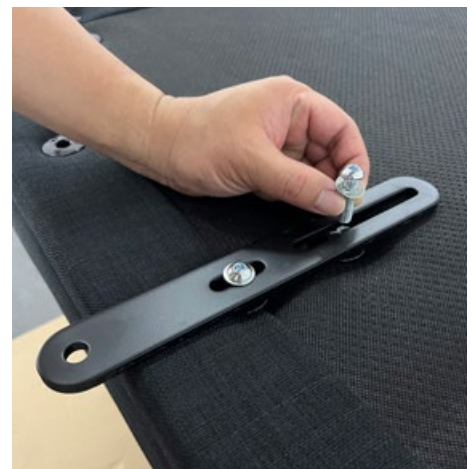

Use two sets of M8 screw (7) and washer (8) to fix one connecting part.

Sales, administration and general enquiries  
**1300 783 783 | customerservice@kcare.com.au | kcare.com.au**

Published July 2024

**KCare**  
Healthcare Solutions



Tighten the screws with 5mm Allen key **(3)**.



Mount the other assist rail connecting part **(2)** by repeating above steps.

## Step 4: Install assist rail.



Place the assist rail **(1)** on top of the mounting holes on the connecting part **(2)**.



Connect assist rail **(1)** with connecting part using two sets of M10 screw **(5)** and big washer **(6)**.



Tighten the two screws with 6mm Allen key **(6)**.

## Step 5: Install the other assist rail by repeating the above steps.

After installation

