

Jannali Bed Bedrail

Installation guide



Installation guide

Step 1: Open bedrail packaging, ensure all parts shown above are included.

Step 2: Locate mounting tubes.

No.	Description	No.	Description
1	Bedrail Long x 1 pcs	4	Screw M8 x 2pcs
2	Bedrail Short x 1 pcs	5	Screw M10 x 2pcs
3	Bedrail mounting parts x 1 set	6	Allen key 6mm x 1 pcs



For short bedrail: Raise headrest to locate the two mounting tubes (1) and (2) on bed frame. **For long bedrail:** Raise headrest and footrest to locate the two mounting tubes (1) and (3) on bed frame

Step 3: Install mounting parts



Insert first bedrail mounting part (the one with metal plate) **(3)** in mounting tube **(2)**. For long bedrail use mounting tube **(3)**. Push it to the end.



Place a M8 screw **(4)** in the mounting hole and use Allen key **(6)** to tighten it firmly.



Install other mounting part **(3)** on mounting tube 1 located at the head end of frame.

Step 4: Install bedrail




Align two mounting holes on bedrail **(1 or 2)** with the mounting parts, slide bedrail in, secure bedrail using M10 screws **(5)**.



The bedrail should look like image below after installation.

Step 5: Fold and unfold bedrail

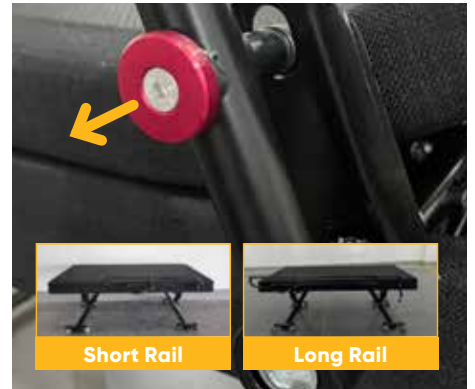

To fold bedrail, put one hand on top of the bedrail **(1 or 2)**, pull out the red locking pin to release the bedrail, and gently put the bedrail down.

Warning: Always hold the bedrail on top to avoid the bedrail falling when lock pin is released. Do NOT put hand under the bedrail when folding it.



To unfold bedrail **(1 or 2)**, bring the folded bedrail up straight until you hear a click which indicates locking pin is engaged. Always double check if the locking pin is engaged properly before use