


# Jannali Bed

## Headboard & Footboard

Installation guide

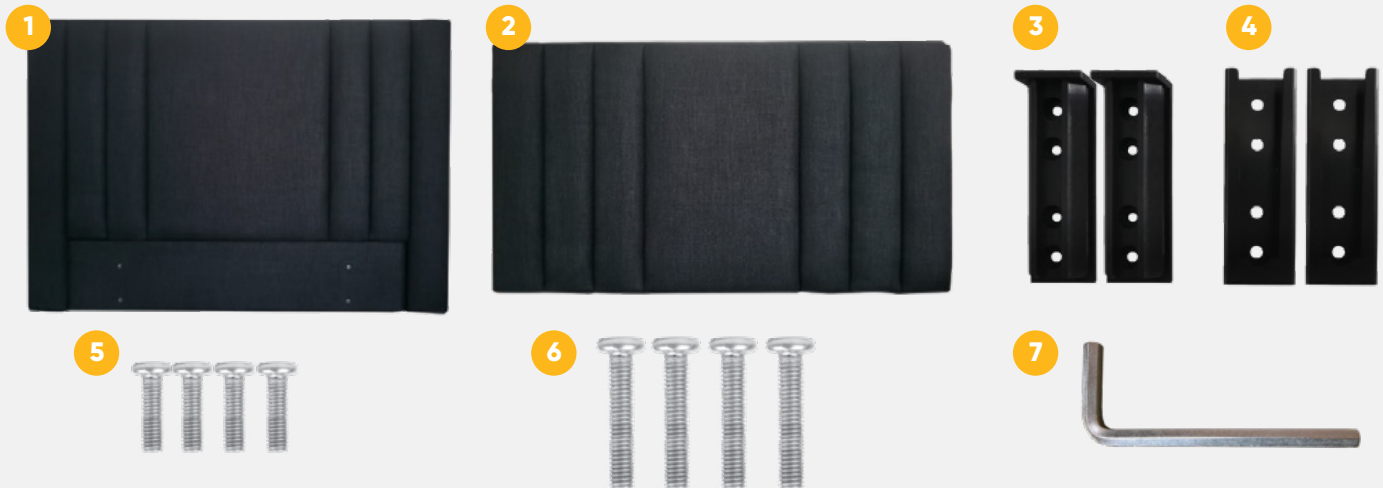

**Note:** For future orders, only long screw will be used.

### Installation guide

**Step 1:** Open headboard/footboard packaging, make sure all parts are included as shown above.

**Step 2:** Install Headboard Mounting Bracket Bed pieces.

No.	Description
1	Headboard
2	Footboard
3	Headboard Mounting Bracket Board piece x 2pcs
4	Headboard Mounting Bracket Bed piece x 2pcs
5	Short Screw M6 x 4pcs
6	Long Screw M6 x 4pcs
7	Allen key 4mm x 1 pcs



Locate mounting holes on bed frame



Place headboard mounting bracket bed piece (4)



Use two long screws (6) to secure the bracket. Install the other bracket (4)

**Step 3:** Install Headboard Mounting Bracket Board pieces. Place two headboard mounting bracket board pieces **(3)** on the headboard mounting holes and use **(5)** two short screws to secure.



**Step 4:** Align mounting brackets on bed board with brackets on frame and slide headboard in.



- Warning:**
- Bed with headboard or footboard fitted should always have enough clearance to wall to use bed tilt function.
    - Clearance headboard to wall when tilt forward: 50mm
    - Clearance headboard to wall when tilt backward: 220mm
  - Periodically inspect screws to ensure they are still tight.
  - Do NOT use headboard or footboard to move the bed.