

# Jannali Bed

# ALIVIO

# Under Bed Light Sensor

Installation guide



## Installation guide

**Step 1:** Open packaging and prepare the assembly kit.

No.	Description
1	Under bed light sensor x 2 pcs
2	Screw M4 x 4 pcs
3	Nut x 4 pcs
4	Allen key x 1 pcs



**Step 2:** Use remote control to raise bed to a comfortable position for installation



Locate mounting bracket on bed frame.

**Step 3:** Place one sensor **(1)** on the bracket and align the mounting holes. Use two sets of screw **(2)** and nut **(3)** to fix the sensor. Use Allen key **(4)** to secure.



**Step 4:** Install the other sensor by repeating the above step.

**Step 5:** Locate the control box from under the bed. Connect sensor **(1)** to control box by plugging the connector to control box sockets as per image below. Either socket can be used, there is no left or right specified.



**Step 6:** Connect the other sensor **(1)** to control box by repeating the above step.

**Note:** If the bed is purchased with this accessory, it may be installed while the bed is still in the box (bed is upside down when it was boxed) for easy access to control box under the bed.