

KCare

User Manual

**Over Toilet
Frame Heavy
Duty Width
Adjustable
3 in 1**

**COMFORT AND
STABILITY**

KA450HDWA
Updated: October 2024

KCare
Healthcare Solutions

Over Toilet Frame Heavy Duty Width Adjustable 3 in 1

KA450HDWA

This over toilet frame has been designed around comfort and stability, for your peace of mind. The height adjustable legs and width adjustable frame ensure that the product fits anywhere and suits your unique needs. The armrests make it both easy and comfortable to stand up out of, and the anti-slip feet ensure your stability while seated or standing. The seat material and removable splash guard make cleaning simple, and effortless. This product can also be used as a shower chair or bedside commode if the bucket is used.

Features

- Ultra-light and extra strong
- Height adjustable legs
- Width adjustable frame
- Comfortable padded seat and armrest
- Removable splash guard for easy cleaning
- Easily fits over most toilets
- Anti-slip tips for stability
- Easy tool-free assembly
- Compact packaging
- Can be used as shower chair or bedside commode
- Bucket with lid included for use as a bedside commode

Specifications

KA450HDWA


Product Net Weight	4.6kg
Overall Height	677-777mm
Overall Width	600-750mm
Overall Depth	465-480mm
Seat Height	500-600mm
Seat Size	460mmW x 415mmD
Width between Side Frame	500-650mm
Maximum User Weight	300kg

Intended Use

This product is designed to be sat upon by someone who has difficulty using a standard toilet.

This product is suitable to be used as a bedside commode or a shower chair.

Components

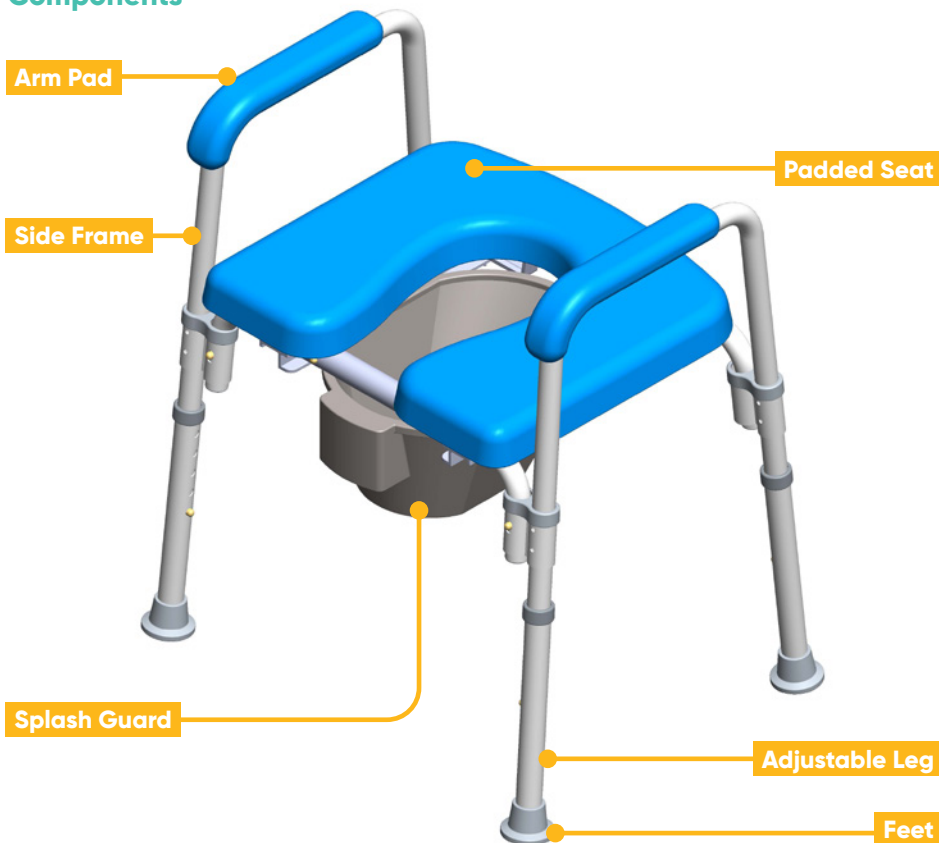

Safety Precautions

Safe Use Environment

- This product should be used over a toilet. It should not be used as regular seating.
- The product is suitable for use in wet areas but not pools and similarly corrosive environments.
- The product is suitable for use in showers or bathrooms.
- This product should only be used in a shower, not a bath.

Safety Warnings

- DO NOT exceed weight capacity of 300kg.
- DO NOT sit on the edge. Always sit in the middle of the seat.
- DO NOT use the product on an uneven surface.
- Ensure all safety pins are fully engaged before each use.
- The product should not be used to stabilize the user when entering or exiting the shower when being used as a shower chair.
- DO NOT lean forwards, backwards or sideways to reach items while seated. This may cause a change in weight distribution and cause the seat to tip.
- While seated the user should have their feet in full contact with the floor.
- Ensure the product is stable on the floor and cannot slide.
- Ensure the items required for toileting or showering are in easy reach when seated.


Warning: Failure to follow these instructions can lead to serious injury or result in death. Misuse of the product may void warranty.

Assembly Instruction

Step 1. Carefully remove all components from packaging.

Ensure all components listed are included in the packaging.

No.	Item
A	Side frame RH
B	Side frame LH
C	Seat assembly
D	Leg x 4
E	Spalsh guard
F	Bucket with lid

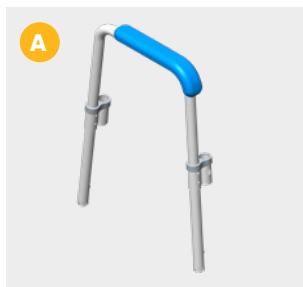

Step 2. Insert each leg into the base of the frame as shown. Ensure that all the legs are adjusted to the same height with the press pin adjustment mechanism.

Step 3. Insert the seat into the frame receiving tubes on one side of the frame, then on the other side, ensuring the safety pins are engaged.

Step 4. Slide the splash guard into the carrier from the front of the seat, ensuring it is the right way around.



Operation Instruction

Height adjustment

To adjust the height of the legs, push in the press pins and slide the leg in or out to reach desired height. Repeat the process for each leg until the product is at the required height.

Width adjustment

To adjust the width of the chair, push in the press pins at front and rear of the seat frame and slide the arm to the desired setting.

Repeat the process on the other side of the frame.



Warning

- Do not attempt to adjust the seat height or width while seated.
- Ensure all the legs are set to the same height before attempting to sit on the seat.
- Ensure the width settings are equal on the left and right side of the seat.
- Check that all safety pins are engaged, refer below image. Failure to do so may result in serious injury.




Accessories



Bucket with lid

Bucket with lid is included for using the product as a bedside commode.

Slide the bucket into the carrier under the seat from the front, ensuring it is the right way around.

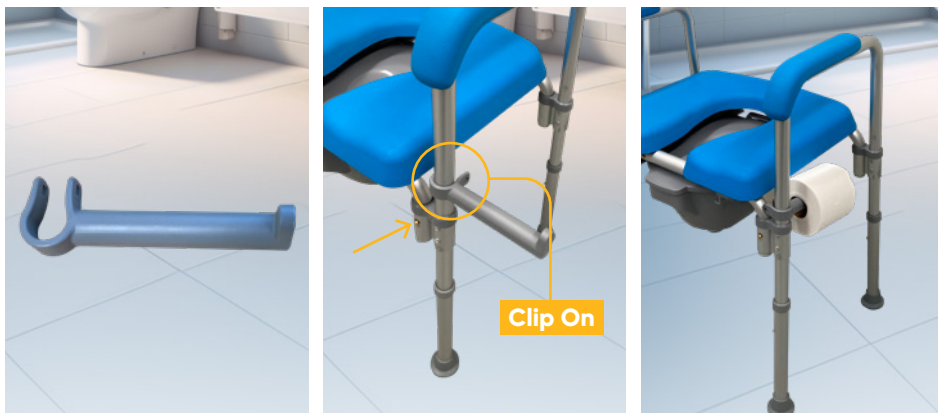

Toilet roll holder

Toilet roll holder is optional for this product.

To install the toilet roll holder, just clip it on to the side frame as per the image shown, and place a toilet roll on the holder.

Maintenance Instruction

Routine Inspections

- Periodically check all pins to make sure they are engaged.
- Check padded seat, armrest pad, splashguard, bucket and foot tips for cracks or damage.

Cleaning

- All surfaces may be cleaned with a damp cloth and mild soap.
- Seat and splash guard may be disinfected.
- DO NOT use harsh chemicals (e.g., alcohol) on any of the surfaces.

Servicing

If you find that your product needs servicing, please contact your dealer for further servicing instructions.

Disposal Instruction

Please dispose of thoughtfully according to local government regulations.

Tested Against Standards

Product tested to AS/NZS 3973:2009 (R2016) Shower/Toilet Chairs (mobile and static) for product safety.

Warranty Against Defects

Warranty for this product falls within K Care standard warranty services. Please see www.kcare.com.au for further warranty information.

Notes

Improving your life everyday™

At Your Service

Sales, administration
and general enquiries

National 1300 783 783

Your Customer Service Team
customerservice@kcare.com.au

kcare.com.au

All information is considered to be true and correct at the date of publication. Changes in circumstances after the time of publication may impact on the accuracy of the information.

